INTENDED USE

The BIONOTE Vcheck cPL 2.0 is an *in vitro* diagnostic test kit for the quantitative measurement of canine pancreas-specific lipase concentration in canine serum. Since the kit provides quantitative measurement of canine serum pancreas-specific lipase levels, BIONOTE Vcheck cPL 2.0 can be tested with every dog presenting with vomiting, anorexia or abdominal pain to diagnose pancreatitis. The BIONOTE Vcheck cPL 2.0 is designed to be used only by veterinarians.

PRINCIPLE

The Vcheck cPL 2.0 Test kit is a fluorescent immunoassay for the quantitative measurement of canine pancreas-specific lipase concentration. The Vcheck cPL 2.0 Test kit uses specific anti-cPL antibodies that bind to cPL. When the specimen is applied to the sample hole of the test device, cPL in the specimen migrates along the nitrocellulose membrane and forms the complexes with anti-cPL antibodies coated on the membrane. As the result, the fluorescence density of the test line reflects the concentration of pancreas-specific lipase in canine serum and plasma. The BIONOTE V check Analyzer reads the density of this test line and calculates the canine pancreas-specific lipase concentration from the calibration curve data. The control line is a reference line which indicates the test has been performed correctly.

MATERIALS PROVIDED

Reagent	10 Tests/Kit
① Vcheck cPL 2.0 test device	10
② Assay diluent tube	10
③ Disposable pipette tip	20
④ Instructions for use	1

MATERIALS REQUIRED, BUT NOT PROVIDED

- BIONOTE Vcheck Analyzer
- 25 μ l pipette
- 100 μ l pipette

STORAGE AND STABILITY

- Store the test kit at 1~30°C. **DO NOT FREEZE.**
- Do not store the test kit in the direct sunlight.
- The test kit is stable until the expiry date that is marked on the package label.

PRECAUTIONS

- The test kit is for canine use only. Do not use for other animals.
- The test device is sensitive to humidity and heat. Perform the test immediately after removing the test device from the foil pouch.
- Do not reuse test components.
- Do not touch the membrane in the result window of the test device.
- Do not use the test kit beyond the stated expiry date marked on the label.
- Do not use the test kit if the pouch is damaged or the seal is broken.
- Do not mix components from different lot numbers, the components in this kit have been quality control tested as a standard batch unit.
- All samples should be handled as being potentially infectious. Wear protective gloves while handling samples. Wash hands thoroughly afterwards.
- Decontaminate and dispose of all samples, used kits and potentially contaminated.

COLLECTION AND PREPARATION OF SAMPLE

- Canine serum should be used with this test.
- [Serum]** Collect the whole blood into a blood collection tube containing NO anticoagulant. Leave to settle for 30 minutes for blood coagulation and then centrifuge to get a serum supernatant.
- If serum samples are not tested immediately, they should be refrigerated at 2~8°C and used within 2 weeks. For longer storage, serum can be frozen (-20°C or colder). Frozen samples should be brought to room temperature (15~30°C) prior to use.

TEST PROCEDURE

All reagents and samples must be at room temperature (15~30°C) before use.

[Coding]

- Turn on V200 Analyzer and select "Standard Test".

- Remove the test device from the foil pouch. Once the "Insert Device" is displayed in the screen, insert the test device.

[Dilution of sample & Measurement]

- Using a 25 μ l pipette, draw 25 μ l of sample and add the sample into an assay diluent tube.

- Use a 100 μ l pipette to mix the sample with diluent by pipetting, for 5~6 times.

- Add the mixed sample (100 μ l) into the sample hole of the test device using the 100 μ l pipette and press the [START] to initiate testing.

* If the time to press [START] button is delayed, it may affect the test result.

- The V200 analyzer will display the test result on the screen after 5 minutes.
- Remove the test device.

INTERPRETATION OF THE RESULT

- Read the concentration value of canine pancreas-specific lipase appearing on the display of the BIONOTE Vcheck Analyzer. (50~2000 ng/mL)
- If "< 50 ng/mL" appears on the display, it means the concentration of canine pancreas-specific lipase in the specimen is less than 50 ng/mL.
- If "> 2000 ng/mL" appears on the display, it means the concentration of canine pancreas-specific lipase in the specimen is greater than 2000 ng/mL.

REFERENCE RANGE

< 200 ng/mL	200~400 ng/mL	> 400 ng/mL
Normal	Suspected	Consistent with pancreatitis

SCREEN MESSAGES AND TROUBLE SHOOTING

[V200]

Error message	Error description
Contaminated Device	The test device is damaged or inserted improperly. Solution: Discard the test device and retest with a new test device and a new specimen.
Insufficient Sample	An insufficient amount of blood has been applied. Solution: Retest with a new test device with enough specimen, ensuring that blood is placed in to the narrow channel in the top edge of the test device.
Expired Device	The test devices are expired. Solution: Retest with a new test device that is not expired.
Temperature Error	The environmental temperature is above or below the operating range of the analyzer. Solution: Move to an area in the acceptable temperature range for the analyzer and perform the test. Do not heat or cool the analyzer artificially.
Printer Connection Fail	The communication between analyzer and barcode or printer has failed. Solution: Reconnect the analyzer and external device. If the error continues after turning ON/OFF the analyzer, please contact BioNote, Inc.
Barcode Error	The measured total hemoglobin is out of the range of 7 to 23 g/dL. Solution: This error occurs when a specimen has a total hemoglobin in the abnormal range. If the error continues after turning ON/OFF the analyzer, please contact BioNote, Inc.
Extremely Total hemoglobin	The test is invalid. Solution: Retest with a new test device and a new patient specimen. If the error continues after turning ON/OFF the analyzer, please contact BioNote, Inc.
Result: Invalid	The calibration is overdue. Solution: If the error continues after turning ON/OFF the analyzer, please contact BioNote, Inc.
Calibration Overdue	Loading a test device that is not supported by the analyzer. Solution: Check whether the test device is manufactured by BioNote, Inc.
Not Supported Device	Internal error has occurred. Solution: If the error continues after turning ON/OFF the analyzer, please contact BioNote, Inc.
EEE	

PERFORMANCE CHARACTERISTIC

- Interference
Measurement was not influenced by interfering substances such as triglyceride, bilirubin. Especially, BIONOTE Vcheck cPL 2.0 can normally measure cPL under condition of less than 10 mg/dL of bilirubin in serum. (reference range of canine bilirubin is 0.1-0.3 mg/dL) However, extremely high bilirubin concentration may interfere with the test result. If the test results indicate "Invalid", we recommend to try a biochemistry assay for diagnosis of other diseases (example, liver disease etc.).
- Measuring Range
50~2000 ng/mL of canine pancreas-specific lipase is measurable in serum.
For > 2000 ng/mL results, samples may be diluted with PBS buffer and measured again for a numerical result. To calculate the final canine pancreas-specific lipase concentration, the measured result must be multiplied by the dilution factor.

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