For veterinary use only

Intended Use

The BIONOTE Vcheck fPL 2.0 is an *in vitro* diagnostic test kit for the quantitative measurement of feline pancreas-specific lipase concentration in feline serum, plasma or whole blood. Since the kit provides quantitative measurement of feline pancreas-specific lipase levels, BIONOTE Vcheck fPL 2.0 can be tested with every cat presenting with vomiting, fever, anorexia, depression or abdominal pain to diagnose pancreatitis. The BIONOTE Vcheck fPL 2.0 is designed to be used only by veterinarians.

Principle

The Vcheck fPL 2.0 Test kit is a fluorescent immunoassay for the quantitative measurement of feline pancreas-specific lipase concentration.

The Vcheck fPL 2.0 Test kit uses specific anti-fPL antibodies that bind to fPL. When the specimen is applied to the sample hole of the test device, fPL in the specimen migrates along the nitrocellulose membrane and forms complexes with the anti-fPL antibodies coated on the membrane. As a result, the density of the test line reflects the concentration of feline pancreas-specific lipase in the feline serum, plasma or whole blood. The BIONOTE Vcheck Analyzer reads the density of this test line and calculates the feline pancreas-specific lipase concentration from the calibration curve data. The control line is a reference line which indicates the test has been performed correctly.

Materials provided

Reagent	5 Tests/Kit	10 Tests/Kit	20 Tests/Kit
① Vcheck fPL 2.0 Test device	5	10	20
② Assay diluent tube	5	10	20
③ Disposable pipette tip	10	20	40
④ Instructions for use	1	1	1

Materials required, but not provided

- BIONOTE Vcheck Analyzer
- 25 µl pipette
- 50 µl pipette
- 100 µl pipette

Storage and Stability

- Store the test kit at 1~30°C. **DO NOT FREEZE.**
- Do not store the test kit in the direct sunlight.
- The test kit is stable until the expiry date that is marked on the package label.

Precautions

- The test kit is for feline use only. Do not use for other animals.
- The test device is sensitive to humidity and heat. Perform the test immediately after removing the test device from the foil pouch.
- Do not reuse test components.
- Do not touch the membrane in the result window of the test device.
- Do not use the test kit beyond the stated expiry date marked on the label.
- Do not use the test kit if the pouch is damaged or the seal is broken.
- Do not mix components from different lot numbers, the components in this kit have been quality control tested as a standard batch unit.
- All samples should be handled as being potentially infectious. Wear protective gloves while handling samples. Wash hands thoroughly afterwards.
- Decontaminate and dispose of all samples, used kits and potentially contaminated materials safely in accordance with national and local regulations.

- In case of using whole blood specimen, it may cause different results between using plasma or serum and using whole blood since each individual has different red cell volume in blood (Hematocrit, HCT)[1]. Hematocrit levels less than 20% may cause falsely high values and hematocrit levels greater than 60% may cause falsely low values.

The use of serum samples is recommended for more accurate results, excluding emergency situation.

*References

[1]Hematology and clinical chemistry parameter in the cat. Mhcael O'Brien, Martin G Murphy, John A Lowe. The Journal of Nutrition, Volume 128, Issue 12, 1998, Pages 2678S-2679S.

- BIONOTE Vcheck analyzer is recommended to use at 15 ~ 30°C.
- A professional veterinarian should make a final diagnosis based on the results of this product, other test result
- ts and clinical findings.

Collection and Preparation of Sample


- Feline serum, plasma, or whole blood should be used with this test.
- [Whole blood]**
 - Blood should be collected with a disposable syringe and added to a tube containing anticoagulant (EDTA).
 - If anticoagulated whole blood is not immediately tested, they should be refrigerated at 2~8°C and used within 24 hours.
 - * Note : Blood samples should not be frozen prior to testing.
 - Severely hemolyzed blood samples may affect the result.
- [Plasma]**
 - Blood should be collected with a disposable syringe and added to a tube containing **anticoagulant (EDTA)**, and then separate plasma by centrifugation.
 - If plasma samples are not tested immediately, they should be refrigerated at 2~8°C and used within 2 weeks. For longer storage, plasma can be frozen (-20°C or colder). Frozen samples should be brought to room temperature (15~30°C) prior to use.
- [Serum]**
 - Collect the whole blood into a blood collection tube containing NO anticoagulant. Leave to settle for 30 minutes for blood coagulation and then centrifuge to obtain a serum supernatant.
 - If serum samples are not tested immediately, they should be refrigerated at 2~8°C and used within 2 weeks. For longer storage, serum can be frozen (-20°C or colder). Frozen samples should be brought to room temperature (15~30°C) prior to use.

Test procedure

All reagents and samples must be at room temperature(15~30°C) before use.

[Coding]


- Turn on V200 Analyzer and select "Standard Test".

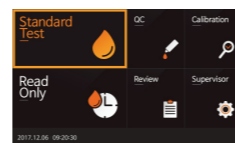- Remove the test device from the foil pouch. Once the "insert device" is displayed in the screen, insert the test device.


- When the [Serum/Plasma] or [Whole blood] menu appears on the display screen, select the menu.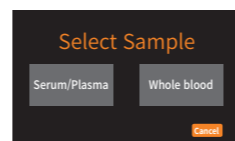

[Dilution of sample & Measurement]


- Use a 50 µl or 25 µl pipette to draw of whole blood 50 µl or serum(plasma) 25 µl and add the sample into an assay diluent tube(100 µl).

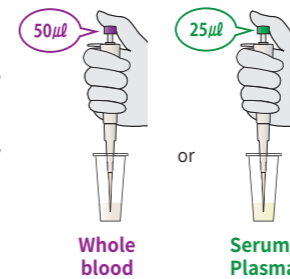
- Use a 100 µl pipette to mix the sample with diluent by pipetting for 5~6 times.

- Add the mixed sample(100 µl) into the sample hole of the test device. Then, press the "Start" button.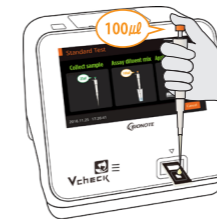

- The V200 analyzer will display the test result on the screen after 15 minutes.
- Remove the test device.

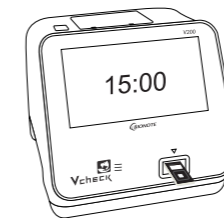Interpretation of the Result

- Read the concentration value of feline pancreas-specific lipase appearing on the display of the BIONOTE Vcheck Analyzer.(1~50 ng/ml)
- If "< 1 ng/ml" appears on the display, it means the concentration of feline pancreas-specific lipase in the specimen is less than 1 ng/ml.
- If "> 50 ng/ml" appears on the display, it means the concentration of feline pancreas-specific lipase in the specimen is greater than 50 ng/ml.

Reference range

Normal	Suspected	Consistent with pancreatitis
≤ 3.5 ng/ml	3.6~5.3 ng/ml	≥ 5.4 ng/ml

* It is recommended that each laboratory establishes its own reference range.

Screen messages and Trouble shooting

[V200]

Error message	Error description
Contaminated Device	The test device is damaged or inserted improperly. Solution: Discard the test device and retest with a new test device and a new specimen.
Insufficient Sample	An insufficient amount of blood has been applied. Solution: Retest with a new test device with enough specimen, ensuring that blood is placed in to the narrow channel in the top edge of the test device.
Expired Device	The test devices are expired. Solution: Retest with a new test device that is not expired.

Temperature Error	The environmental temperature is above or below the operating range of the analyzer. Solution: Move to an area in the acceptable temperature range for the analyzer and perform the test. Do not heat or cool the analyzer artificially.
Printer Connection Fail	The communication between analyzer and barcode or printer has failed. Solution: Reconnect the analyzer and external device. If the error continues after turning ON/OFF the analyzer, please contact BioNote, Inc.
Barcode Error	The measured total hemoglobin is out of the range of 7 to 23g/dL. Solution: This error occurs when a specimen has a total hemoglobin in the abnormal range. If the error continues after turning ON/OFF the analyzer, please contact BioNote, Inc.
Extremely Total hemoglobin	The test is invalid. Solution: Retest with a new test device and a new patient specimen. If the error continues after turning ON/OFF the analyzer, please contact BioNote, Inc.
Result: Invalid	The calibration is overdue. Solution: If the error continues after turning ON/OFF the analyzer, please contact BioNote, Inc.
Calibration Overdue	Loading a test device that is not supported by the analyzer. Solution: Check whether the test device is manufactured by BioNote, Inc.
Not Supported Device	Internal error has occurred. Solution: If the error continues after turning ON/OFF the analyzer, please contact BioNote, Inc.
EEE	

Performance characteristic

- Interference
Measurement was not influenced by interfering substances such as triglyceride, bilirubin, hemoglobin and cholesterol.
- Measuring Range
1~50ng/ml of feline pancreas-specific lipase is measurable in serum, plasma, and whole blood. For >50ng/ml result, samples may be diluted with PBS buffer and re measured for a numerical result. To calculate the final feline pancreas-specific lipase concentration, the measured result must be multiplied by the dilution factor.

- Result comparison with Commercial Product

Manufacturer	Sample Cases	Result Correlation
Commercial fPL test vs. Vcheck fPL 2.0	n=102 serum samples	R ² =0.9621

- Correlation between Specimen Types

Test Specimen Type	Number of Samples	Correlation
Serum vs. Plasma (EDTA treated)	n=47 samples	R ² =0.9898
Serum vs. Whole Blood (EDTA treated)	n=47 samples	R ² =0.9052*

* The difference of each individual HCT value may cause little deviation of results between whole blood and other sample types. (See "Precaution 10").

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